

Length deficit / Rotationplasty

Example 1



Example 2



Example 3



With Your® Liner


Limb



3D model



Your® Liner

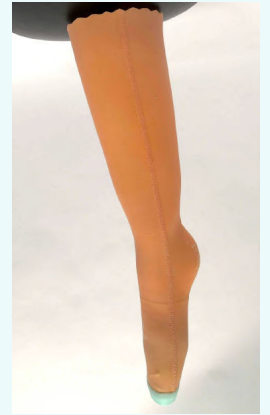

Indication

Ankle joint has been substituted to knee joint through a Rotationplasty procedure (Van Nes / Borggreve rotation). One limb is much shorter than the other one, requiring use of an orthoprosthesis.



Typical issues

No suitable off-the-shelf liner available

Off-the-shelf liners are designed for common amputation cases. Rotationplasty residual limbs exhibit shapes and functions for which off-the-shelf liners are unadapted (e.g. no suitable size, inadapted compressions, unadapted thickness profiles, etc.)



Key features



Anatomically tailored



Adapted compressions



Your® Liner

With its anatomically tailored shape, adapted compressions, and possibility to customise the local thicknesses, Your® Liner allows to provide a silicone interface which uniquely matches the specificities and needs of rotationplasty stumps.

How can Your® Liner help ?

